



Garment Measuring Guide

HUNTLEIGH


A MEMBER OF THE ARJO FAMILY

Garment Measuring Guide

The following guide is intended as an aid to help the user select the correct size garment.

When determining the correct size garment:

- Remove any bandaging, stockings or bulky dressings
- Measure the points as shown on the diagram below
- DO NOT pull the measuring tape tight
- Make sure the garment extends beyond the area under treatment.

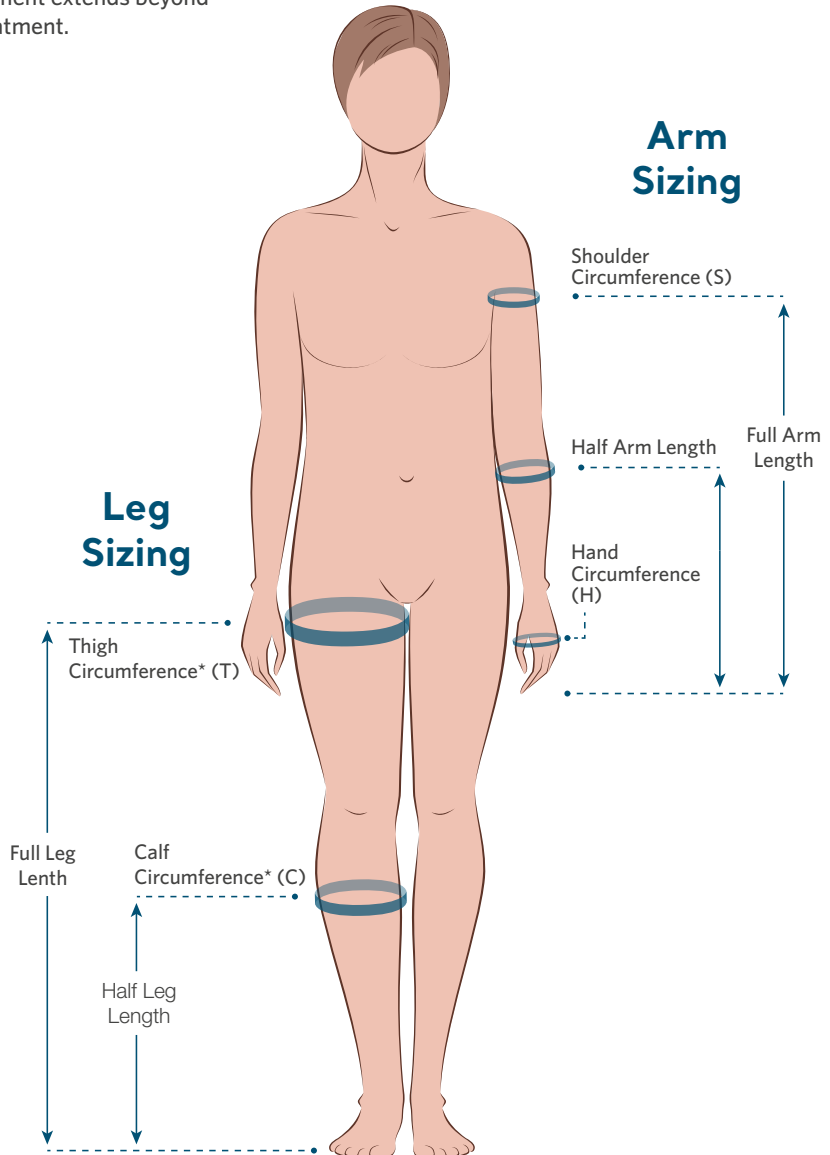

*Measurements should be made at the widest point on the calf and thigh.

Lymph Assist (Hydroven 12) Arm

Twelve Chamber Arm Garments



Order Code	Type	Length (L)	Shoulder Circumference (S)	Hand Circumference (H)
316A68	Full Arm	68cm	62cm	44cm
316A78	Full Arm	78cm	62cm	44cm

Lymph Assist (Hydroven 12) Arm Insert

Twelve Chamber Arm Inserts



Order Code	Type	Length (L)	Additional Width at Shoulder	Additional Width at Hand
316AI68	Full Arm	68cm	17cm	13cm
316AI78	Full Arm	78cm	17cm	13cm

Lymph Assist (Hydroven 12) Leg

Twelve Chamber Leg Garments



Order Code	Type	Length (L)	Thigh Circumference (T)	Calf Circumference (C)
316L76S	Standard Leg	76cm	71cm	59cm
316L84S	Standard Leg	84cm	71cm	59cm
316L76W	Wide Leg	76cm	79cm	62cm
316L84W	Wide Leg	84cm	79cm	62cm

Lymph Assist (Hydroven 12) Leg Insert

Twelve Chamber Leg Inserts



Order Code	Type	Length (L)	Additional Width at Thigh	Additional Width at Foot
316LI76	Leg Insert	76cm	19cm	14cm
316LI84	Leg Insert	84cm	19cm	14cm

As a proud member of the Arjo family, we have been committed to supporting healthcare professionals in improving outcomes and enhancing patient wellbeing since 1979. We do this through our proven solutions for Vascular Assessment & Treatment and Fetal & Patient Monitoring. With innovation and customer satisfaction as our guiding principles, we strive for clinical excellence and improved performance, for life.

Manufactured and distributed by Huntleigh Healthcare Ltd.
35 Portmann Moor Road, Cardiff, CF24 5HN, United Kingdom
T: +44 (0)29 20485885 sales@huntleigh-diagnostics.co.uk
www.huntleigh-diagnostics.com

Registered No: 942245 England & Wales. Registered Office:
ArjoHuntleigh House, Houghton Hall Business Park, Houghton Regis, Bedfordshire, LU5 5XF
©Huntleigh Healthcare Limited 2019

A Member of the Arjo Family

® and ™ are trademarks of Huntleigh Technology Limited

As our policy is one of continuous improvement, we reserve the right to modify designs without prior notice.

AW: 1001043-2



HUNTLEIGH

LIT 779384/EN-5